

DAW[®] SUCTION SOCKET SHEATH





Claim to fame

- Protects the soft tissues of the brim area, especially in the medial aspect of the residual limb
- Reduces or eliminates irritation, bruising and abrasions
- Facilitates the proper donning of the prosthesis

DAW provides a solution even for the **suction socket** wearer who has a friction/abrasion problem in the proximal medial area of the residual limb. This exclusive design will not impair suction when properly fitted.

HOW TO DETERMINE THE RIGHT SIZE, TWO EASY STEPS:

1. Measure the external circumference of the proximal socket brim follow the diagonal, lateral to medial (**Measure the socket not the limb**).
2. Read the corresponding size on the following chart.

| Circumference  | 12 3/4 in to 15 3/4 in (33 to 40 cm) | 15 7/8 in to 19 3/4 in (41 to 50 cm) | 19 7/8 in to 23 5/8 in (51 to 60 cm) | 23 3/4 in to 27 5/8 in (61 to 70 cm) | 27 3/4 in to 32 in (71 to 81 cm) |
|--|---|---|---|---|-------------------------------------|
| Size  | F2 | F3 | F4 | F5 | F6 |

HOW TO ORDER

Stock #: 14-Ø7 Fx

x = Size based on the chart above




DAW[®]
INDUSTRIES


© Copyright 2015, DAW Industries, San Diego, CA. All Rights Reserved.

Orders (800) 252-2828 • Technical Hotline (877) 242-2423 • www.daw-usa.com


4 EASY STEPS FOR PROPERLY FITTING THE DAW SUCTION SOCKET SHEATH

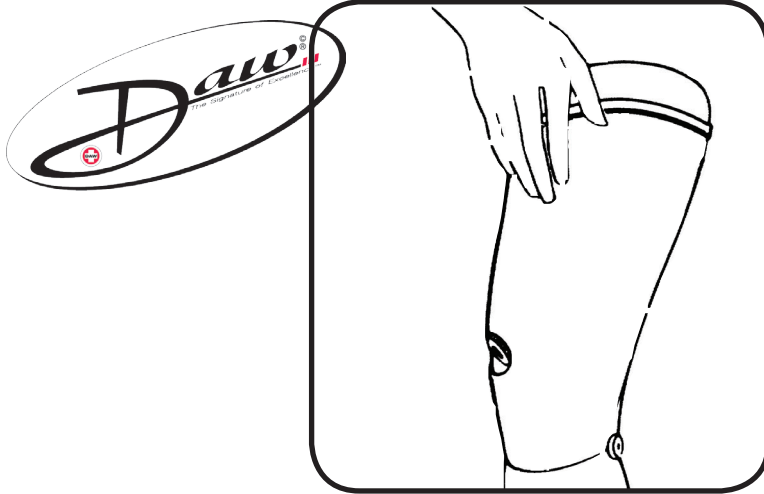


1 Apply the self adhesive foam strip around the socket brim, externally, and following the contour of the brim (approximately 1 ½ inches below the top of the socket).

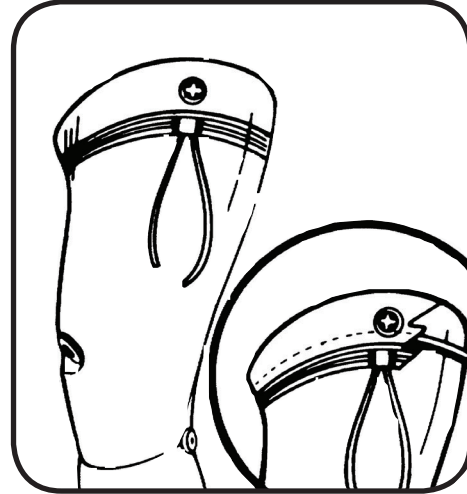

2 Fold the prosthetic sheath over the socket brim of the prosthesis so that the elastic band is below the self-adhesive foam strip. Tie the string on the lateral side. Tuck the knot under the elastic band.

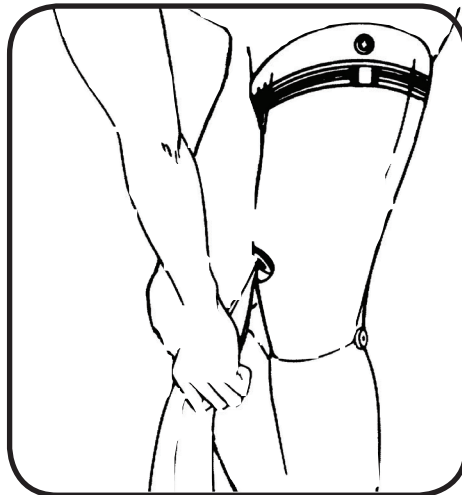

3 Insert the residual limb in the socket in the usual manner with the DAW Slipper™, the DAW EZ-IN™, or a pull sock.

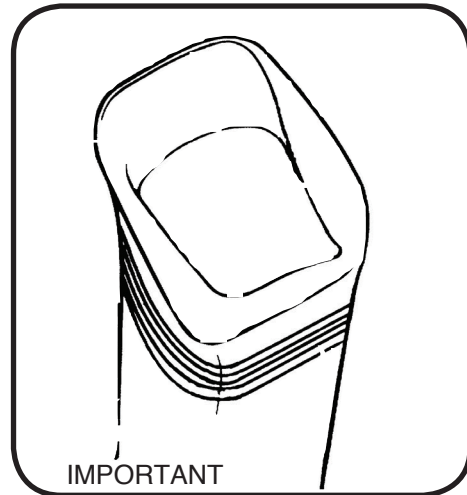

4 Should the suction not be secure, the sheath is too far down in the socket. To avoid this, go back to step 1 and move the self-adhesive foam strip further down.



DAW INDUSTRIES

© Copyright 2015, DAW Industries, San Diego, CA. All Rights Reserved.

Orders (800) 252-2828 • Technical Hotline (877) 242-2423 • www.daw-usa.com

