

# DAW SLIPPER

## “THE TUG OF WAR IS OVER”

### CLAIM TO FAME

- Slip into your suction socket smoothly and naturally.
- Avoid tissue misplacement, the common curse of traditional donning methods.
- Eliminate tissue damage and skin irritation.
- Ensure ideal donning and perfect positioning in the suction socket every time.
- The ultimate complement to state-of-the-art comfort with OPTIFlex2™ flexible sockets.

The ideal solution for the Above-Knee Suction Socket Wearer. Any other means of donning a properly fitting AK socket encourages misplacement of tissues. This method seems to resemble the traditional “pull-sock” technique which involved strong hand pulling. The DAWSlipper’s material is extremely slippery and therefore slides over itself. The donning MUST BE done WITHOUT weight bearing, the limb will slip into place effortlessly.

\* DAWSlipper is a DAW INDUSTRIES copyrighted trademark.

### HOW TO ORDER

Stock #: D-1Ø7-6FX

x = size (Please Refer to Chart on Back)



**DAW**  
**INDUSTRIES**


© Copyright 2015, DAW Industries, San Diego, CA. All Rights Reserved.

Orders (800) 252-2828 • Technical Hotline (877) 242-2423 • [www.daw-usa.com](http://www.daw-usa.com)


# DAW SLIPPER

## “THE TUG OF WAR IS OVER”


**1**  
Measure the circumference of the residual limb at the ischial level. Select correct size from chart below.

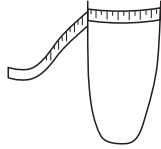


**4**  
Feed the “slip-off” string through the valve hole.



**2**  
Don the *DAWSlipper* on the residual limb the way you would with a DAW prosthetic sock.

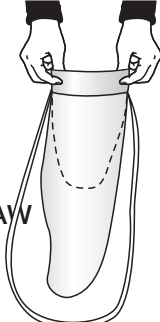


**5**  
Slip down residual limb into the socket – This will happen smoothly due to the very slick nature of the *DAWSlipper*. The tissue contained by the *DAWSlipper* will slide in easily. As the limb slides in, the *DAWSlipper* slides over itself and off the limb via the valve hole.

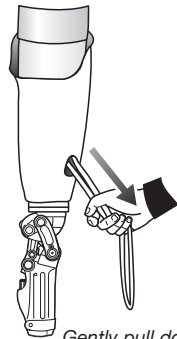


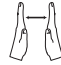
*Gently pull down – No tugging*

**3**  
Fold the top of the *DAWSlipper* over itself (approx. 4" down). The “slip-off” string must be on the outside of the *DAWSlipper*.

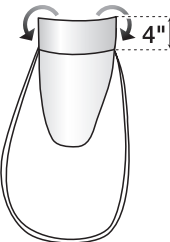

**Important : Do not weight bear during this procedure.**

### HOW TO ORDER

 ISCHIAL CIRCUMFERENCE	10 to 12 <sup>3</sup> / <sub>4</sub> in (25 to 32cm)	12 <sup>3</sup> / <sub>4</sub> to 15 <sup>3</sup> / <sub>4</sub> in (32 to 40cm)	15 <sup>7</sup> / <sub>8</sub> to 19 <sup>3</sup> / <sub>4</sub> in (41 to 50cm)	18 <sup>7</sup> / <sub>8</sub> to 23 <sup>5</sup> / <sub>8</sub> in (51 to 60cm)	23 <sup>3</sup> / <sub>4</sub> to 27 <sup>5</sup> / <sub>8</sub> in (61 to 70cm)
 SIZE	F1	F2	F3	F4	F5
Stock # 6 (Package of 6)	D-107-6 F1	D-107-6 F2	D-107-6 F3	D-107-6 F4	D-107-6 F5
<b>Assorted Kit: D107-6 K</b>	Qty. 1, size F2 + Qty. 2, size F3 + Qty. 2, size F4 + Qty. 1, size F5				

The most common sizes are F3 and F4. Due to the material soft elasticity in length, one length fits all.



**DAW INDUSTRIES**