

DAW'NER™

EDEMA REDUCER

ULTIMATE CARE NON-STOP



CLAIM TO FAME

- No pulling of scarred tissues.
- Guarantees proper gradient pressure time after time.
- Eliminates the need of nightly “rebandaging”.
- Easy to use for arthritic hands.



The **DAW'NER** is the indispensable tool to apply the **DAWREDUCER** without disturbing the sensitive tissues of the new Amputee – By prestretching the Edema Reducer over the **DAW'NER**. The Residual Limb is introduced into the **DAW'NER** and the Edema Reducer softly slides into place without stretching or pulling the patient’s tissues.

HOW TO ORDER

Stock #:
Small
SSD-L
6" DIAMETER



DAW
INDUSTRIES

DAW'NER APPLICATION:

1 Insert **DAWREDUCER** inside **DAW'NER**.

2 With both hands stretch **DAWREDUCER** over upper edge of **DAW'NER**.

3 Pull **DAWREDUCER** down until the end is tight across top of **DAW'NER**.

4 Place residual limb in **DAW'NER**. Make sure suspension strap is on lateral side of leg.

5 Pull **DAW'NER** upward allowing Edema Reducer to slide onto leg.

6 Remove **DAW'NER** and adjust suspension strap around waist.



ULTIMATE CARE – NON-STOP:

DAW is proud to provide you with the ultimate edema reduction system, designed specifically to prepare your limb for the best prosthetic care.

IMPORTANT: This DAW'NER and DAWREDUCER must be prescribed and fitted by a Certified Prosthetist.



DAW
INDUSTRIES

