

Success

In Every K2 Step

Feather-Lite™ Foot

Feather-Lite for your K2 patients combines total safety with natural ultra-light ease of Gait.

- 4 Resistance settings for both Dorsi and Plantar flexion
- 2 different toe levers: Firm or Natural
- Feather-Lite (260g to 420g)
- Max Patient Weight 300lb (135kg)
 - Standard Ankle - 220lb
 - HW Ankle - 300lb
- Sizes 22-28 R/L
- Build height - Pyramid 115/16 in (5cm)
 - Multi-Axis 33/4 in (9.4cm)

Adjustable Multi-Axis Ankle



Options:
Firm or Natural
Roll-Over



L-Codes:

K2 Pyramid: L5972

K2 Multi-Axis: L5972+L5986

Feather-Lite Item #:

Pyramid: TFK2N/F-PYRCAU/LBKxxR/L

Multi-Axis Original: TFK2N/F-MACAU/LBKxxR/L

Multi-Axis HW: TFK2HWN/F-MACAU/LBKxxR/L



DAW®
Industries

Orders (800) 252-2828 • Technical Support (800) 242-8669 • www.daw-usa.com

© Copyright 2019, DAW Industries, San Diego, CA - All Rights Reserved 020819

K2 Feather-Lite™ Foot

Instructions to place/replace anterior/posterior bumpers



Feather-Lite Foot
w/ Natural Keel →

3 Different Bumper Durometers:


Green = Soft
Yellow = Medium
Red = Hard

(Additional bumpers available upon request)



NOTE: Foot comes preassembled. Medium Loctite was applied to the Foot Nut. The Foot Nut was then properly torqued to 108in/lbs (12.2 Nm). Should you wish to replace the bumpers, it is suggested that you heat up the Foot Nut to allow for its removal. Re-assembly requires medium Loctite and proper torque.

Feather-Lite Foot
w/ Firm Keel

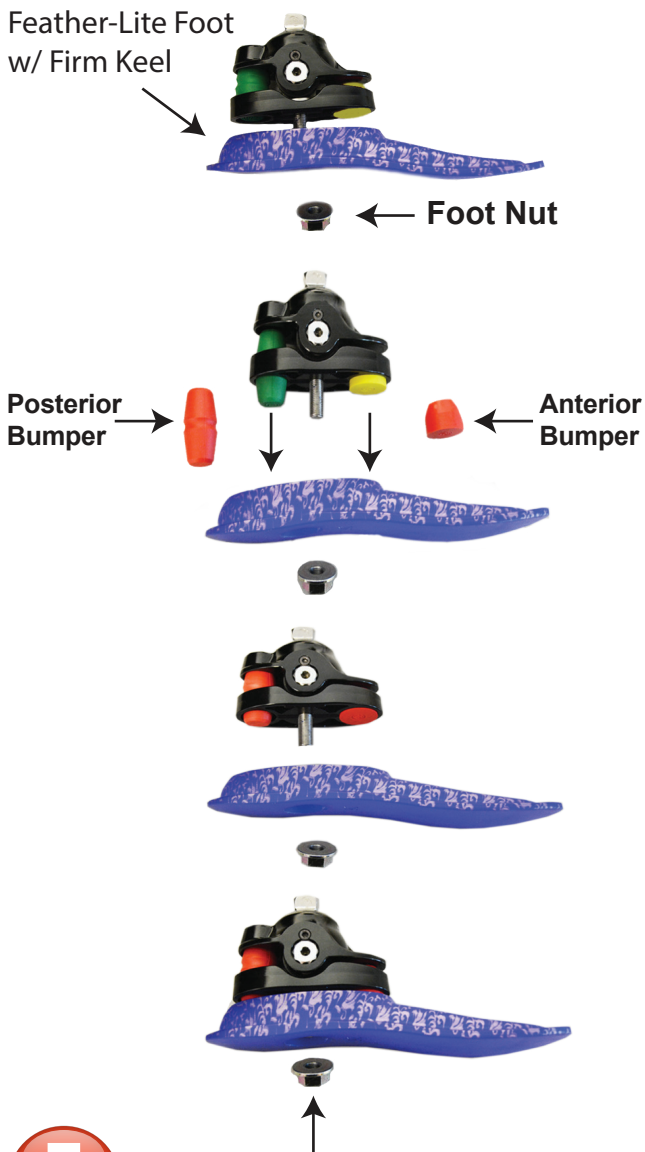

1. Remove the Foot Nut from the multi-axis ankle

Please use a 17mm socket.

Do not use **Vise Grips**.

2. Replace anterior and/or posterior bumpers

3. Reassemble ankle keel with Foot Nut

4. Apply medium Loctite to bolt threads. Be sure to tighten Foot Nut to 108 in/lb (12.2 Nm)