PROSTHESIS**





Claim to fame

- Two-way elastic construction
- Controlled elasticity around the thigh
- Superior breathability for minimal perspiration
- Tight grip around prosthesis
- Easy to apply
- Available in regular and long



The Fix Prosthesis™ is the ideal, non-suction, way to provide the BK amputee with a comfortable, secure and dependable prosthetic suspension. The two way elastic allows for good air circulation and freedom of knee movement. The proximal elastic band is contoured to prevent constriction of blood flow while the distal band provides a strong compression to guarantee a tight grip on the prosthesis.

HOW TO ORDER

Regular Length	Long
Stock # : DØ5 - Ø1 xx	Stock # : DØ5 - Ø1L xx

xx = size (Please Refer to Sizing on Back)





- a) Available in long length for extra suspension and comfort.
- b) Portion covering the thigh The controlled elasticity in width prevents blood vessels' constriction.
- C) Portion covering the knee The two-way elasticity allows for freedom of knee movement.
 - d) Portion covering the prosthesis The strong elastic compression guarantees a tight grip on the prosthesis.

HOW TO DETERMINE THE RIGHT SIZE

- 1. Measure the circumference of the limb in centimeters. 5 centimeters above the center of the knee.
- 2. The size will be the next even figure. (i.e.: if circumference is 33cm, the size is 34).
- 3. Sizes available: 28 to 56 (even numbers only).

