


CLEANING THE SHOWER SEAL SLEEVE


The **SHOWER SEAL-SLEEVE** should be washed regularly.

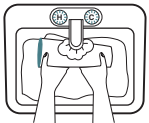

1. After removing the *Shower Seal Sleeve*, it will be inside-out (gel side out). Hand wash the *Shower Seal Sleeve* with soap & water. You **MUST** use a soap with neutral PH that does not contain: Alcohol, solvent, coloring or fragrance (An example in the USA is Unscented Ivory Soap). **DO NOT** submerge the *Shower Seal Sleeve* in water and **DO NOT** scrub.



2. Rinse the *Shower Seal Sleeve* very thoroughly in COLD water. Again, **DO NOT** immerse *Shower Seal Sleeve*.

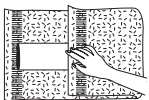

3. Gently pat dry the *Shower Seal Sleeve*. **DO NOT** rub.

4. Keep *Shower Seal Sleeve* inside-out (cloth side in). Let dry overnight.

CLEANING INSTRUCTIONS FOR THE TIBIAL SHOWER SEAL



- For general cleaning, the *Shower Seal Tibial Covering* can be cleaned with soap & water using a clean soft cloth. Do not use chlorine, acetone or abrasive detergents.
- In case of marks or stains, use **DAWSKIN Cleaner™**.

Skin Allergy

The Shower Seal Sleeve is manufactured with great care from the highest grade of TPE material. The material is relatively inert compared to other liner materials. Allergic reactions occur in less than one in one hundred thousand people. Most skin redness is not an allergy. Should you develop skin redness and/or irritation, contact your Prosthetist immediately.

Warranty

The durability of the Shower Seal Sleeve depends on many factors such as patient activity level and smoothness and design of socket brim. Therefore Shower Seal Sleeve can only be warranted against manufacturer defects.

Return Policy

Only a Certified Prosthetist can obtain a Return Authorization Number (RA#) from DAW. Products returned to DAW without an RA# will be returned to sender.



DAW®
3 Industries

www.daw-usa.com

© Copyright 2019, DAW Industries, San diego, CA - All Rights Reserved 020819

AMPUTEE

Patent Pending

INSTRUCTION MANUAL

Be Safe!
Shower with confidence on your own 2 legs!

Shower Seal™

“Anti-slip”
Waterproof

TABLE OF CONTENTS

The Story About This Amazing Innovation.....	1
Donning the Shower Seal Sleeve	2
Quick Fix Patches	2
Cleaning the Shower Seal Sleeve.....	3
Cleaning Instructions for The Tibial Shower Seal.....	3



DAW®
Industries

www.daw-usa.com

© Copyright 2019, DAW Industries, San diego, CA - All Rights Reserved 020819

THE STORY BEHIND THIS AMAZING INNOVATION

Dear Amputee,

I remember in my childhood seeing my father David, a World War II BK amputee, hopping around on one leg to take a shower in our family bathroom. I also remember hearing on occasion a thundering noise... the scary sign of my father slipping and falling down! Thank God he was never seriously hurt.

Then I thought it was just a normal occurrence. No more! With DAW's waterproof anti-slip Shower Seal, every amputee can now navigate on both feet as safely as a non-amputee in any wet environment (bathroom, swimming pool, beach, etc.).

Our Patent Pending Daw Shower Seal comes in many skin tones to be less conspicuous in any surroundings. Here is a convenient selection swatch for the Shower Seal which will help all you BK amputees live a safer daily life and feel whole again.



DAW®
1 Industries


www.daw-usa.com

© Copyright 2019, DAW Industries, San diego, CA - All Rights Reserved 020819

DONNING THE SHOWER SEAL SLEEVE

The *Shower Seal Sleeve* is designed to ROLL UP (DO NOT PULL UP) easily onto your patient's residual limb. Therefore, **DO NOT** apply any type of lubricant to either the residual limb or the *Shower Seal Sleeve* prior to donning. Any oil based product such as petroleum jelly will damage the *Shower Seal Sleeve* and should not be used. The residual limb should be clean, dry, and free of soap residue and/or talcum.

1. Stretch open the bottom (small) opening of the *Shower Seal Sleeve* and fit it over the upper brim of the socket so that it is at least 2 in (5cm) below the top of the socket brim, all around the brim, even in the back part of the leg (popliteal area).
2. ROLL the *Shower Seal Sleeve* over itself all the way below the socket brim.
3. Make sure the seam of the *Shower Seal Sleeve* is positioned on either the right or the left side of the leg.
4. Don the residual limb into the prosthetic socket.
5. ROLL back up (DO NOT PULL UP) the *Shower Seal Sleeve* over the knee & thigh.

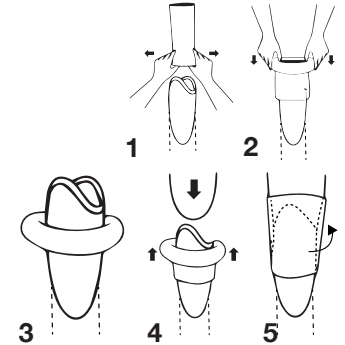

- IMPORTANT:**
- Always ROLL the *Shower Seal Sleeve* onto the residual limb. Never pull up. Pulling up will result in stretching the patient's skin and tissues creating an uncomfortable fit.
 - Avoid air pockets and wrinkles between the residual limb and the *Shower Seal Sleeve*.

Storing the Shower Seal Sleeve

When the liner is not being worn it should be stored in a cool dry place with the cloth side out.

QUICK FIX PATCHES

Two self-adhesive patches are provided in the unlikely event that the patient pokes a hole in the Tibial Shower Seal. If the patches are applied in the foot or ankle areas, for proper bonding, be sure to wait overnight before any motion occurs.

Clean the Tibial Shower Seal per instructions on page 3.

How to apply: Peel off the film and apply the self adhesive patch over the hole.



DAW®
Industries

www.daw-usa.com

© Copyright 2019, DAW Industries, San diego, CA - All Rights Reserved 020819