




DAW Industries

Cool Liner™

E-Z Transtibial Sizing Selection Chart

DISTAL CIRCUMFERENCE

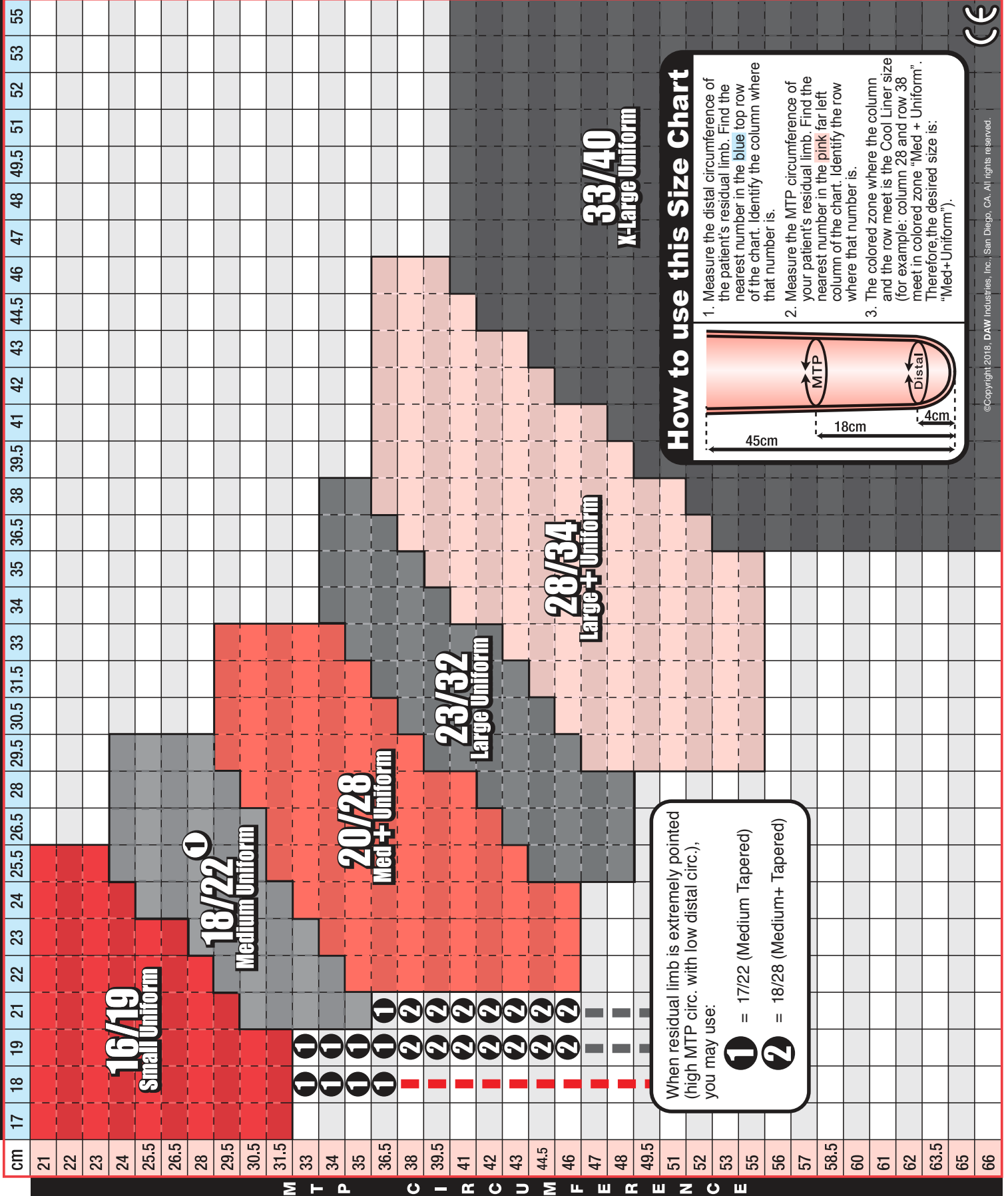

How to use this Size Chart

1. Measure the distal circumference of the patient's residual limb. Find the nearest number in the blue top row of the chart. Identify the column where that number is.
2. Measure the MTP circumference of your patient's residual limb. Find the nearest number in the pink far left column of the chart. Identify the row where that number is.
3. The colored zone where the column and the row meet is the Cool Liner size (for example: column 28 and row 38 meet in colored zone "Med + Uniform". Therefore, the desired size is: "Med+Uniform").

When residual limb is extremely pointed (high MTP circ. with low distal circ.), you may use:

1 = 17/22 (Medium Tapered)
2 = 18/28 (Medium+ Tapered)



©Copyright 2018, DAW Industries, Inc., San Diego, CA. All rights reserved.